
PARI BOY SX
PARI JuniorBOY SX
PARI TurboBOY SX

Type 085

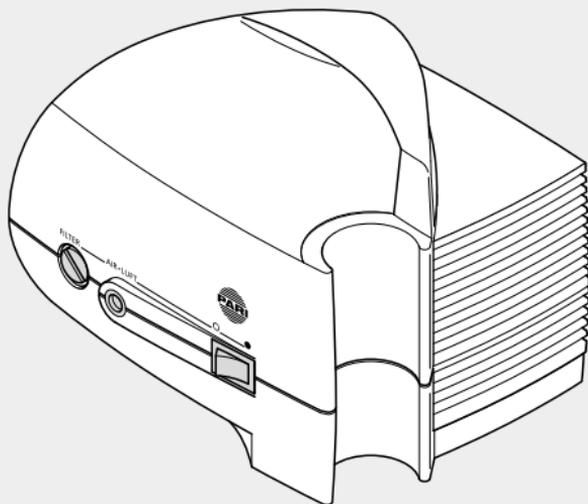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en

Instructions for use

Information as of: April 2017. All rights reserved. Technical and design changes and printing errors reserved. Principle shown in illustrations.

IMPORTANT INFORMATION	29
General.....	29
Information about the instructions for use	29
Structure of safety instructions	29
Using the compressor	30
Treatment of babies, children and anyone who requires assistance	31
PRODUCT DESCRIPTION	32
Components	32
Intended purpose	32
Intended use.....	32
Contraindications.....	33
Product variants and combinations	33
Working parts	34
Operating life	34
PREPARATION FOR USE	35
Setting up the compressor	35
Preparing the treatment.....	37
Performing treatment.....	38
Ending the treatment	40
MAINTENANCE	41
Cleaning	41
Replacing the filter.....	42
Repairing	43
Storage.....	43
TROUBLESHOOTING	44

TECHNICAL DATA	45
General compressor data	45
Classification according to DIN EN 60601-1	45
Information about electromagnetic compatibility	46
Ambient conditions	46
MISCELLANEOUS	47
Disposal.....	47
Terms and conditions of warranty	48
Explanation of symbols	49
Contact	50

1 IMPORTANT INFORMATION

1.1 General

Please read these instructions for use carefully and completely. Do not discard, so you can consult them at a later date. If you fail to comply with the instructions for use, injury or damage to the device cannot be ruled out.

In addition, follow the instructions for use of the nebuliser and the accessories that are being used.

1.2 Information about the instructions for use

If these instructions for use are lost, you can request another copy from PARI GmbH [see: Contact, page 50]. The instructions for use for some products can be retrieved on the internet in both German and English. Simply visit: www.pari.com (on the respective product page).

1.3 Structure of safety instructions

Safety-critical warnings are categorised according to hazard levels in these instructions for use:

- The signal word **WARNING** is used to indicate hazards which, without precautionary measures, can result in serious injury or even death.
- The signal word **CAUTION** is used to indicate hazards which, without precautionary measures, can result in minor to moderate injury or impair treatment.
- The signal word **NOTICE** is used to indicate general precautionary measures which are to be observed to avoid damaging the product during use.

1.4 Using the compressor

The compressor is an electrical device that is powered by supply voltage. It has been designed so that no live parts are accessible. However, if ambient conditions are unfavourable or if the compressor or power cord is damaged, this protection may no longer be assured. Therefore, please follow the instructions below to avoid damage to the device and the associated danger of contact with live parts (e.g., electric shock):

- The compressor is not splashproof. Protect the device and the power cord from liquid and moisture. Never handle the device, the power cord or the power plug with damp hands. In addition, the device must not be operated in a damp environment.
- Only use the PARI power cord to operate the compressor.
- Make sure that the local supply voltage complies with the voltage data marked on the compressor identification label.
- Never leave the compressor unattended while it is in use.
- For safety reasons, always disconnect the power plug from the socket under the following circumstances:
 - if a malfunction occurs during operation
 - before cleaning and maintaining the device
 - immediately after use
- Always pull the power plug, not the cable, to disconnect the power cord from the socket.
- Make sure that the power cord is never kinked, pinched or jammed. Do not pull the power cord over sharp edges.
- Keep the compressor and the power cord away from hot surfaces (e.g., stove top, electric fire, open fire). Direct heat may damage the compressor housing or the power cord insulation.
- Keep the power cord away from domestic animals (e.g., rodents). They may damage the power cord insulation.

- The compressor must not be operated and/or the power plug must be unplugged from the socket immediately if the compressor or the power cord is damaged, or if a fault is suspected after the device has been dropped or suffered a similar shock.

1.5 Treatment of babies, children and anyone who requires assistance

Babies, children and anyone who requires assistance must be supervised constantly by an adult during inhalation therapy. This is the only way to ensure safe and effective treatment. Such persons often underestimate the hazards involved (e.g., strangulation with the power cord or the connection tube), thus resulting in a risk of injury.


The product contains small parts. Small parts can block the airways and lead to a choking hazard. Therefore, make sure that you always keep the compressor, the nebuliser and the accessories out of the reach of babies and infants.

The device and the connection tube at the compressor outlet can become very warm during normal operation. Therefore, particularly with individuals who require special assistance, ensure that the user's skin does not come into contact with the compressor. Prolonged contact may cause burns, particularly for patients with impaired temperature perception.

2 PRODUCT DESCRIPTION

2.1 Components

Check that all of the components belonging to your compressor are included in the package. If anything is missing, please notify the dealer from whom you purchased the PARI compressor immediately.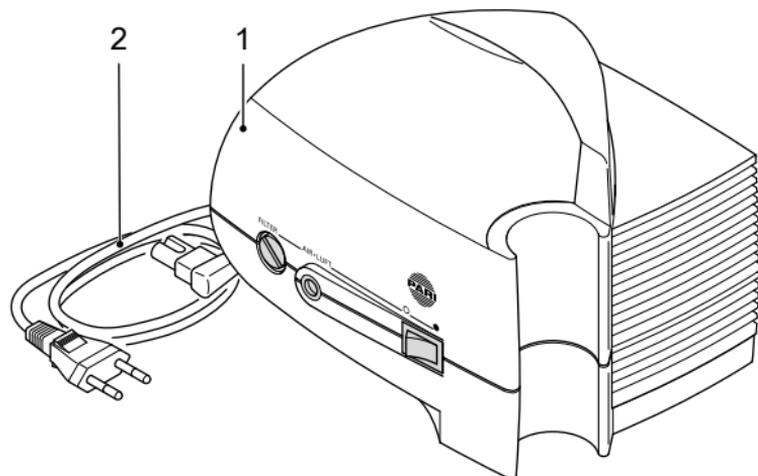

- (1) Compressor
- (2) Power cord

2.2 Intended purpose

The purpose of the compressor is to generate compressed air for operating a PARI nebuliser.

2.3 Intended use

The compressor must only be used for PARI nebulisers. It is intended exclusively for indoor use and can be operated by the patient himself/herself.

Only original PARI nebulisers and PARI accessories must be used.

2.4 Contraindications

There are no contraindications known to PARI GmbH.

2.5 Product variants and combinations

There are several different versions of the PARI BOY SX compressor:


- PARI JuniorBOY SX (compressor coloured yellow)
- PARI TurboBOY SX (compressor coloured blue)
- PARI BOY SX (compressor coloured blue)

The compressor is intended for operation with a PARI nebuliser.

Separate instructions for use accompany all nebulisers, whether they are included in the basic equipment or purchased subsequently.

2.6 Working parts

The compressor includes the following working parts:

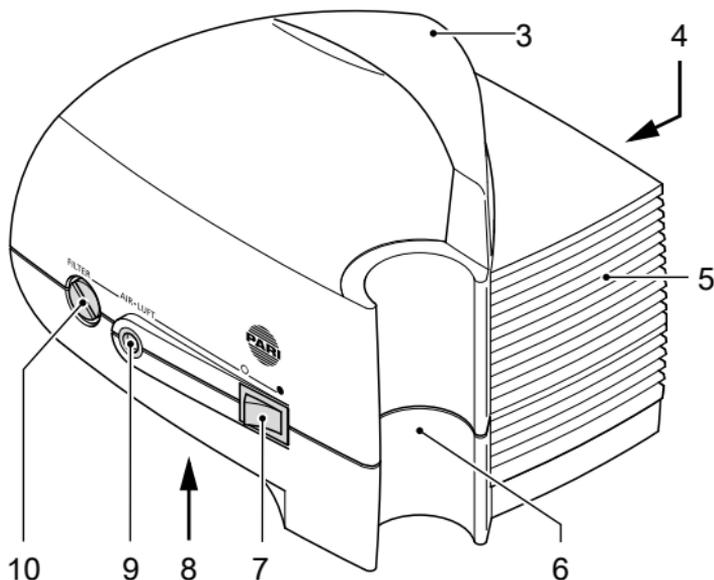- (3) Carrying handle
- (4) Power cord socket
- (5) Ventilation slits
- (6) Holder for nebuliser
- (7) On/off switch
- (8) Identification label (bottom of device)
- (9) Air outlet
- (10) Filter

2.7 Operating life

The compressor has an average expected operating life of:

- approx. 1,000 operating hours
- max. 5 to 6 years

3 PREPARATION FOR USE

3.1 Setting up the compressor

WARNING

Risk of fire due to a short circuit

A short circuit in the device can cause a fire. In order to reduce the risk of fire in such an event, follow the instructions below:

- Do not operate the compressor close to readily flammable objects such as curtains, tablecloths, or paper.
- Do not operate the compressor in areas where there is a risk of explosion or in the presence of oxidising gases (e.g., oxygen, nitrous oxide, flammable anaesthetics).

CAUTION

Quality of treatment impaired by electromagnetic interference

Electrical devices can cause electromagnetic interference. Interference can impair the function of the devices and thus also the effectiveness of the treatment.

- Do not place the PARI device immediately beside or on top of other devices.
- Keep at a minimum distance of 30 cm from portable wireless communication devices (including accessories therefor, such as antenna cables or external antennas).

 **CAUTION**

Risk of injury from tripping or caused by the device falling

A compressor placed in a poor position or for which the cables are routed untidily poses a risk of injury.

- Route the power cord in such a way that it is not a tripping hazard and no one can become entangled in it.
- Do not place the compressor above head height and make sure that it cannot be pulled down by the power cord or the connection tube.

NOTICE

Danger of device fault caused by falling

If the compressor falls, it may be damaged so badly inside and outside that it is no longer usable.

- Place the compressor on a firm, flat, dry surface. A soft base such as a sofa, a bed or tablecloth is not suitable.

NOTICE

Danger of device fault caused by dust

If the device is operated in a very dusty atmosphere, more dust may collect inside the housing and cause a fault in the device .

- Do not operate the compressor on the floor, under the bed or in workshops.
- Only operate the compressor in a low-dust environment.

The room in which the compressor is operated must satisfy certain conditions [see: Operation, page 46]. The compressor is not intended for outdoor use.

3.2 Preparing the treatment

WARNING

Danger of death by electrocution from a faulty device!

If the device has a fault, a person may come into contact with live parts, resulting in the danger of electric shock, for example.


- Check the compressor housing, the power cord and the power plug for damage every time before use.
- Only use the compressor if all components are intact.

CAUTION

Quality of treatment impaired by electromagnetic interference

The use of third-party products can result in higher levels of electromagnetic emissions or lower the resistance of the PARI device to electromagnetic interference.

- Use only original spare parts and original accessories from PARI.
- Assemble the nebuliser as described in the instructions for use of the nebuliser.
- Insert the connection tube of the PARI nebuliser with a slight twist into the air outlet on the compressor.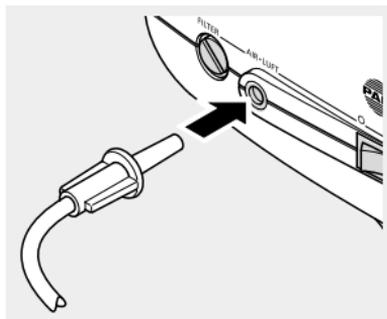


- Insert the nebuliser in the holder on the compressor intended for this purpose.
- Fill the nebuliser as described in the instructions for use of the nebuliser.

⚠ WARNING

Danger of electrocution due to damp hands

Liquids can conduct electricity, thereby posing a risk of electric shock.

- Never take hold of the power cord or the power plug with damp or wet hands.
- Connect the power cord to the power cord socket of the compressor.
- Plug the power plug into a suitable socket.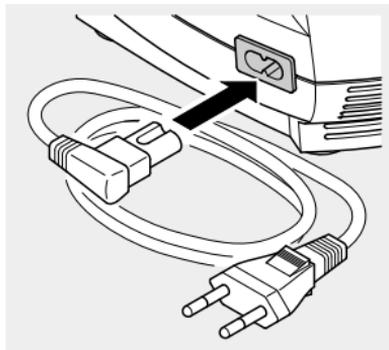

3.3 Performing treatment

⚠ WARNING

Danger of death by mixing up tubes

When several tubing systems are present, there is a possible danger of death if different connection options are confused with each other. This danger applies particularly **in the case of patients in need of constant care** who must receive artificial nutrition or other infusion treatments, for example.

- Check carefully to ensure that the other end of the PARI connection tube that is connected to the compressor is attached to the PARI nebuliser.

⚠ CAUTION

Quality of treatment impaired by electromagnetic interference

If electrical devices are placed too close to each other, the function of both can be impaired due to electromagnetic interference.


- If the PARI device has to be placed immediately beside or on top of other devices for operation, all devices must be monitored to ensure that they are working properly.

NOTICE

Danger of device fault due to overheating


Air must be able to circulate freely around the compressor to dissipate the heat generated in the housing.

- Never operate the compressor while it is in a bag.
 - Never cover the compressor while it is operating.
 - Make sure that the ventilation slits on the compressor are unobstructed at all times while it is operating.
-
- Before starting the treatment, pull the nebuliser out of the holder.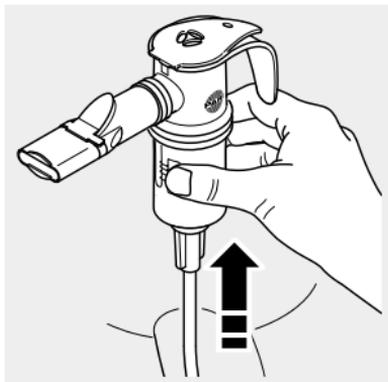

- Switch the compressor on.

⚠ WARNING! Danger of death by electrocution from a faulty device! If the device has a fault, a person may come into contact with live parts, resulting in the danger of electric shock, for example. Switch the compressor off and disconnect the power plug from the mains socket immediately if a fault is suspected (e.g., following a fall or if there is a smell of burning plastic).

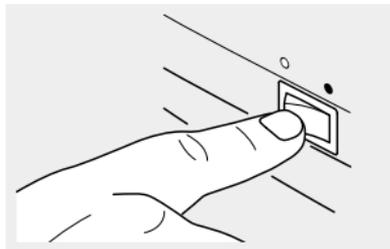- Check that an aerosol is being generated before you start with the inhalation.
- Perform the inhalation as described in the instructions for use of the nebuliser.

3.4 Ending the treatment

- Switch the compressor off.
 - Place the nebuliser back in its holder.
 - Disconnect the power plug from the mains socket.
- i** *Complete disconnection from the mains is only certain when the power plug has been unplugged from the socket.*

4 MAINTENANCE

Do not attempt to carry out any maintenance on the device unless it is switched off and the power plug has been disconnected from the socket.

4.1 Cleaning

WARNING

Danger of electrocution

Liquids can conduct electricity, thereby posing a risk of electric shock.


- Always switch the compressor off and disconnect the power plug from the mains socket before starting to clean it.

NOTICE

Danger of device fault due to liquid penetration

If liquids get into the interior of the device, this may cause a fault in the device.

- Never spray any liquids onto the compressor or the power cord.
- Wipe the outer surface of the housing with a clean, damp cloth.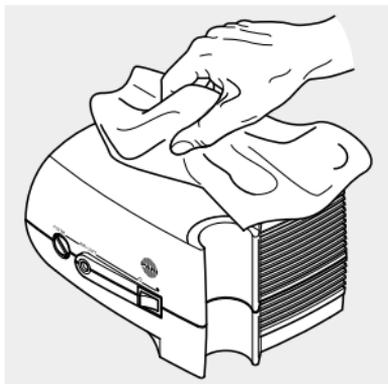

In order to clean the nebuliser, the connection tubing and the accessories, follow the notes in the respective instructions for use.

If liquid does get into the compressor, the device must not be used under any circumstances. Before using the compressor again, you must contact the PARI GmbH Technical Service department or Service Center.

4.2 Replacing the filter

NOTICE

Danger of device fault due to incorrect air filter

If air filters that are not designed for the compressor are used, the compressor may be damaged.

- Only use air filters provided by PARI for use with your compressor.


In normal operating conditions, the compressor filter must be replaced after every 200 operating hours, but at least once a year.

Also please check the filter at regular intervals (after every 10 applications). If it is soiled (grey or brown colour) or blocked, replace it. The filter must also be replaced with a new one if it has become damp.

The filter cannot be cleaned and then reused!

To replace the filter, proceed as follows:

- Unscrew the filter holder from your compressor using a suitable coin.
- Pull the old filter out of the filter holder and fit the new filter in its place.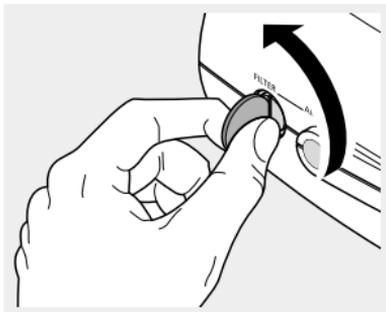

- Reinsert the filter holder together with the new filter in your compressor and screw it tight with the coin.

4.3 Repairing

The compressor must only be serviced by the PARI GmbH Technical Service department or a service centre expressly authorised by PARI. If the compressor is opened or manipulated by anyone else, all claims under the warranty shall be void. In these cases, PARI GmbH will also accept no liability.


4.4 Storage

NOTICE

Damage to the power cord by kinking

If the power cord is wound or bent very tightly, the wires inside the cable may break. This will render the power cord unusable.

- Do not wind the power cord around the compressor.
- Disconnect the power cord from the compressor and wind it up.
- Clip the power cord below the carrying handle for storage.

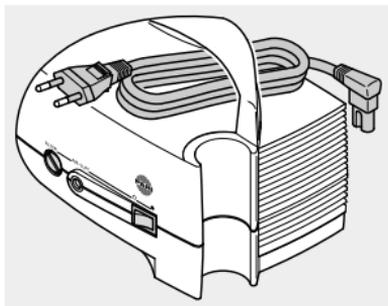Store the compressor and accessories safely out of continuous direct sunlight.

Always disconnect the compressor from the mains for storage because electrical devices that remain plugged in present a potential hazard source.

5 TROUBLESHOOTING

In the event of faults that are not listed in this chapter, or if the suggested procedure does not correct the fault, contact the PARI GmbH Service Center.

Fault	Possible cause	Procedure
The compressor does not start up.	The power cord is not plugged into the device properly.	Check that the power cord is seated correctly in the compressor power cord socket.
	The power plug is not plugged into the mains socket properly.	Check that the power plug is seated correctly in the mains socket.
No medication is coming out of the nebuliser.	The nebuliser nozzle is blocked.	Clean the nebuliser.
	The connection tube is not connected properly.	Check that both ends of the tube are seated properly.
	The connection tube is leaking.	Replace the connection tube.

6 TECHNICAL DATA

6.1 General compressor data

Supply voltage [V]	see back cover page
Mains frequency [Hz]	
Power consumption [A]	
Housing dimensions (W × H × D)	19.2 × 14.5 × 15 cm
Compressor flow ¹	approx. 5.1 l/min
Sound pressure level ²	approx. 56 dBA
Pressure	approx. 1.6 bar
Weight	1.7 kg

6.2 Classification according to DIN EN 60601-1

Type of electric shock protection	Protection class II
Degree of protection from electric shock of the part used (nebuliser)	Type BF
Degree of protection against water ingress in accordance with EN 60529 (IP rating)	No protection
Degree of protection when used in the presence of flammable mixtures of anaesthetics with air, with oxygen, or with nitrous oxide	No protection
Operating mode	Continuous operation

1) Towards PARI nebuliser nozzle (Ø 0.48 mm)

2) Maximum A-evaluated sound pressure level of the compressor (in accordance with DIN EN 13544-1, section 26)

6.3 Information about electromagnetic compatibility

Electrical medical equipment is subject to special precautionary measures with regard to electromagnetic compatibility (EMC). Such equipment must only be installed and operated in accordance with EMC instructions.

Portable and mobile high-frequency communication devices can disrupt electrical medical equipment. Using accessories, converters and power cords other than those specified (with the exception of converters and power cords that the manufacturer of the medical electrical device sells as spare parts for internal components) can result in higher emission levels or lower the resistance to interference of the device.

The device must not be placed directly beside or on top of other devices for operation. If the medical electrical device must be placed beside or on top of other devices to operate it, it should be monitored constantly to ensure that it is operating properly in the arrangement used.

Technical data on electromagnetic compatibility (EMC information) is available in table format upon request from PARI GmbH or on the internet at the following linked page:

<https://www.pari.com/fileadmin/Electromagnetic-compatibility-1.pdf>

6.4 Ambient conditions

Operation

Ambient temperature	+10 °C to +30 °C
Relative humidity	30% to 75% (non-condensing)
Atmospheric pressure	700 hPa to 1,060 hPa

Use of the device in professional healthcare facilities is limited to the inpatient wards and the intensive care unit. Use of the

device in areas with elevated magnetic or electrical radiation (e.g., close to an MRI scanner) is not permitted.

The device is intended for operation in healthcare situations at home. It is not permitted to operate it in trains, motor vehicles or aeroplanes.

Transportation and storage

Minimum ambient temperature (without monitoring of relative humidity)	-25 °C
Maximum ambient temperature (with relative humidity of up to 93%, non-condensing)	+70 °C
Humidity	max. 93%
Atmospheric pressure	500 hPa – 1,060 hPa

7 MISCELLANEOUS

7.1 Disposal

This product falls within the scope of the European Council Directive on Waste Electrical and Electronic Equipment (WEEE)³. Accordingly, this product must not be disposed of with domestic waste. The disposal regulations prevailing in the respective member countries must be observed (e.g., disposal by local authorities or dealers). Materials recycling helps to reduce the consumption of raw materials and protect the environment.

3) Directive 2012/19/EU of the EUROPEAN PARLIAMENT AND THE EUROPEAN COUNCIL of July 4, 2012 on waste electrical and electronic equipment.

7.2 Terms and conditions of warranty

PARI guarantees that your device, if used according to the instructions, will be free from defects in material and workmanship caused by the manufacturing process for the warranty period indicated on the warranty certificate, beginning on the date of initial purchase. Claims under the warranty shall be subject to a limitation period of 12 months. The warranty provided by PARI applies in addition to the warranty obligation of your dealer. Your statutory rights with respect to your dealer in the event of defects are not limited by the warranty or any claim under the warranty. The warranty certificate stamped by the dealer serves as your proof of warranty and ownership.

What does the warranty cover?

If, exceptionally, a defect is discovered, PARI will at its discretion repair or replace the device, or refund the purchase price of the product. If it is replaced, the replacement device may either be the same model or a model that is at least comparably equipped. Replacement or repair of the device shall not serve as the basis for a new warranty. All replaced old devices or parts shall become the property of PARI. Further claims are excluded. This applies particularly for any claims for compensatory damages. This disclaimer of warranty shall be ineffective in the event of injury to life, limb or health, in cases of wilful wrongdoing and gross negligence, product liability and if substantive obligations under the warranty agreement are violated.

The warranty shall be cancelled if

- the device has been operated or used improperly with respect to the descriptions in the instructions for use
- damage is present that is attributable to the effects of water, fire, lightning, or the like
- the damage was caused by transporting the device incorrectly or a falling impact
- the device has been misused or not cared for correctly

- the serial number on the device has been changed, removed, or otherwise rendered illegible
- repairs, adaptations or modifications have been made to the device by persons not authorised by PARI

Moreover, the warranty does not cover wearing parts, that is to say device parts that are exposed to normal wear.

In the event of a complaint, please bring the entire device to your specialist dealer or send it to us packed in the original box, postage paid, together with the warranty certificate stamped by the dealer.

The "warranty period" begins on the date of purchase.

7.3 Explanation of symbols

The following symbols can be found on the compressor and/or the packaging:

	Please follow the instructions for use.
	Serial number of the device
	Off / On
	Alternating current
	Protection class of the part used: Type BF
	Protect device from moisture
	Protection class II device

	Minimum and maximum temperatures for transport and storage
	Minimum and maximum humidity
	Minimum and maximum air pressure
	The medical device was distributed commercially after 13 August 2005. This product must not be disposed of with normal domestic waste. The symbol of the refuse bin with a cross through it indicates that it must be disposed of separately.
	Manufacturer
	This product satisfies the requirements of 93/42/EEC (Medical devices) and 2011/65/EU (RoHS).

7.4 Contact

For all product information and in the event of defects or questions about usage, please contact our Service Center:

Tel.: +49 (0)8151-279 279 (German-speaking)
+49 (0)8151-279 220 (international)

PARI BOY SX
PARI JuniorBOY SX
PARI TurboBOY SX

Type 085

CERTIFICATE OF GUARANTEE

We guarantee this device for 4 years, commencing on the date of purchase.

Application
no:

Bar code label

Confirmation of purchase:

The appliance with the above appliance number was
sold in its original packaging.

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©2017 PARI GmbH Spezialisten für effektive Inhalation, 085D3002-B 08/17